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**MEISINGER** special instruments and assortments for Endodontia offers the dentist all possibilities from root canal treatment to root fillings with titanium pins. Because of this wide choice, all current endodontic methods of treatments can be carried out easily and safely.

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#### MEITRAC II Endo Safety System

#### MEITRAC III Endo Safety System

#### MEITAN Titanium Root Filling Post System

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Root canal reamers "Gates", stainless

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Root canal reamers "Peeso", stainless

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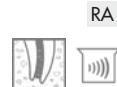
The Wuerzburg Post is available in two different versions:

1. **Post-and-core**: To be used for fixed partial dentures. This core offers a preformed six-degree conical abutment which can be individualized at your discretion in analogy to the classic core-buildup.
2. **Ball-end**: This version serves as a reliable attachment for removable dentures utilizing the conventional 2.25 mm ball-ended abutment to be used in connection with your preferred construction elements, offering angular compensation and short leverage.

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- The abutment can be placed without an implant on the natural root
- Wider indications: Teeth with strongly curved root shapes can be used
- Safer to use: Greatly reduced risk of leakage and perforation
- More effective: Time-saving and simple application to one-piece construction
- Increased stability: Highly rigid post transmits forces evenly into dentine

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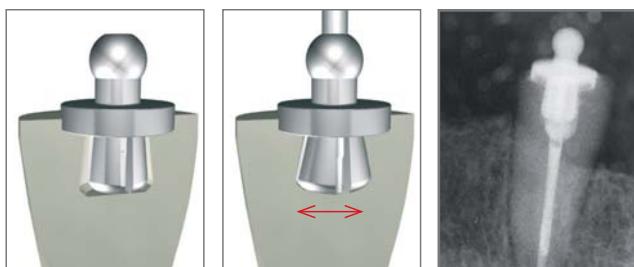
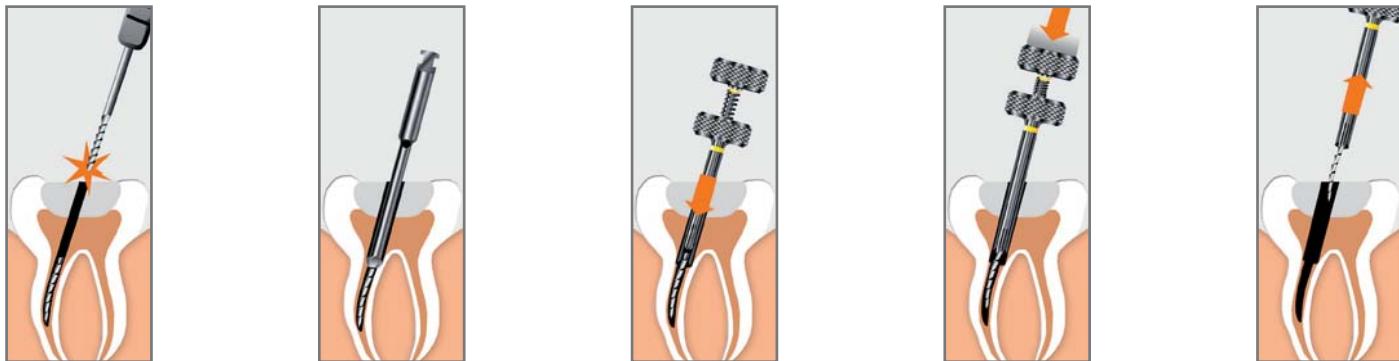
With this endodontic safety-system, broken fragments can be grasped and removed successfully, precisely, and quickly, e.g. root canal instruments, root filling pins, silver points, and root-posts from the root canal. These systems are designed in such a way that fragments with diameters of 0.15 to 0.50 mm (MEITRAC I), 0.55 to 0.90 mm (MEITRAC II), as well as 0.95 to 1.50 mm (MEITRAC III) can be removed safely.

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- Flexible systems for secure removal of fragments from the root-canal
- Deployable in removal of fragments from root-canal instruments, silver points, root-filling pins and root-posts
- Controlled and easy handling
- Allows for minimally invasive procedure to preserve sound dental tissue from being damaged

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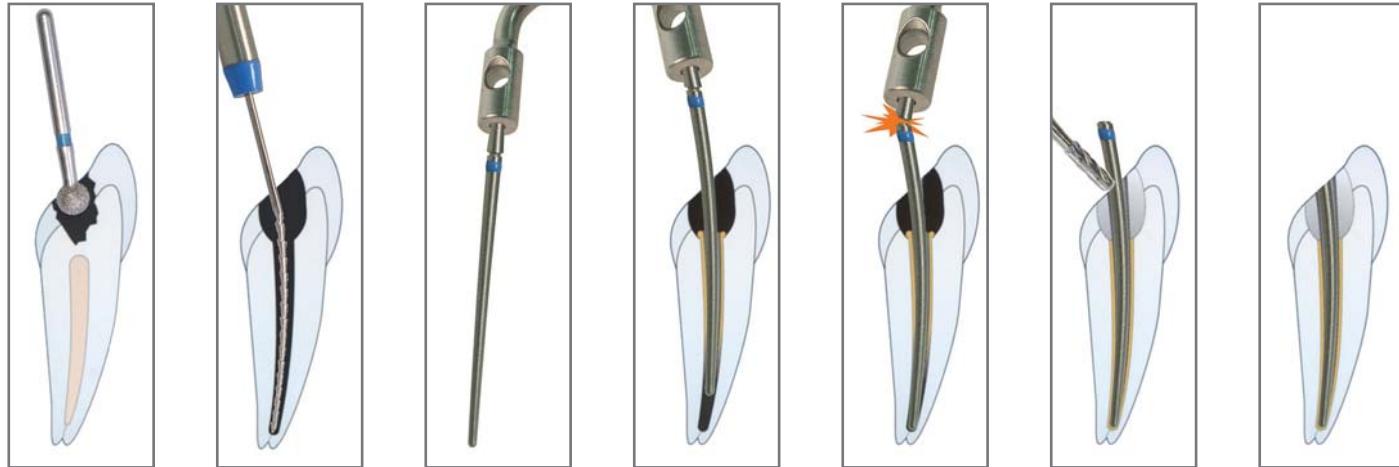
Maximum reliability, less material

This titanium root filling post system seals the root canal optimally without requiring an additional root filling.

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- Bioinert structural-osteotropic root filling post, biocompatible
- Dimensionally stable
- Easy to handle
- Reduces your workload considerably
- Placed with instruments and condensed ultrasonically
- The root post is easily re-examined
- Less complications due to excessive condensing
- Radiopaque
- Stabilizes the root and crown – no additional post and core required
- Few instruments required

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